

INSTRUCTIONS FOR USE

FOR YOUR SAFETY:

- ▲ Maximum weight limit is **600 pounds**.
- ▲ Read and follow all instructions before use.
- ▲ Do not use if ramp is damaged or unstable.
- ▲ It is important that you refer to your equipment's (wheelchair or scooter) User Guide for the proper degree of incline/decline and chair direction before attempting ramp use. **Never exceed its recommendations.**
- ▲ Use ramp only with a qualified helper and always use your lap belt.

PACKAGE CONTENTS:

AR808 1" Tall Threshold Ramp

(A) foot, (H) flap, ¼" screws

AR810 1.5" Tall Threshold Ramp

(A) foot, (B) 1½" extension, (H) flap, ¼" screws

AR812 2" Tall Threshold Ramp

(A) foot, (C) 2" extension, (H) flap, ¼" screws

AR816 3" Tall Threshold Ramp

(A) foot, (C) 2" extension, (D) 3" extension, (H) flap, ¼" screws

AR824 4" Tall Threshold Ramp

(A) foot, (C) 2" extension, (D) 3" extension, (E) 4" extension, (H) flap, ¼" screws

AR830 5" Tall Threshold Ramp

(A) foot, (B) 1½" extension, (E) flap, ¼" screws

AR836 1" Tall Threshold Ramp

(A) foot, (C) 2" extension, (E) flap, ¼" screws


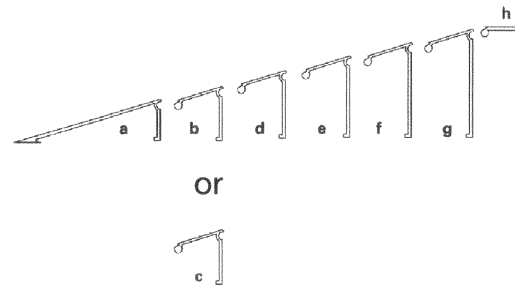


FIGURE 2

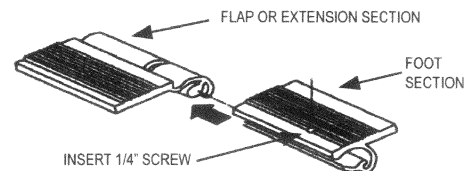

SINGLE RAMP APPLICATION

STANDARD THRESHOLD & LANDING

- a. Slide each section together and insert screw as shown in (figure 2). Assemble the ramp by connecting the individual sections as shown in (figure 1).
- b. Place the ramp against the threshold or landing (figure 1). The adjustable flap should rest on your threshold or landing to provide a smooth surface for your wheels. If the adjustable flap is not used, simply place ramp against the edge of the door runner and attach.
- c. Attach your ramp to the threshold or landing using the holes in either the foot section or the flap. Wood screws, sheet metal screws, or concrete bolts may be necessary for this method. It is also possible to use double face tape or caulking to attach the ramp. Foam rubber or other protective material may be put on the bottom of the legs to protect certain floor surfaces.

MULTI-RAMP APPLICATIONS

UNEVEN THRESHOLDS OR SLIDING DOORS

There are situations where a second Threshold Ramp may be needed on the inside of the threshold. If so, then the purchase of a second Threshold Ramp of equal or different height will be necessary. Follow instructions for single ramp application when installing second ramp.

For sliding doors, the flap will not be used. Simply place leg of ramp against edge of door runner and attach per item 'c' in SINGLE RAMP APPLICATION section.